

**DOCOBOAPP™**  
Sending photo  
messages from  
the device

## Sending photo messages from the device

To send a photo to a clinician from the Docobo Remote Monitoring device, follow the steps below:

1. Select the messaging icon from the home screen
2. Select the camera icon in the messaging screen


**Note: if you cannot see this please contact technical support and they can assist you with activating this feature**

3. Line up the camera with the item you wish to take a photo of

Make sure the picture is in good lighting and clear

You may need to wipe the lens with a clean cloth or tissue to make sure the image is as clear as possible

5. Once happy with the image you want to send press 'Send'

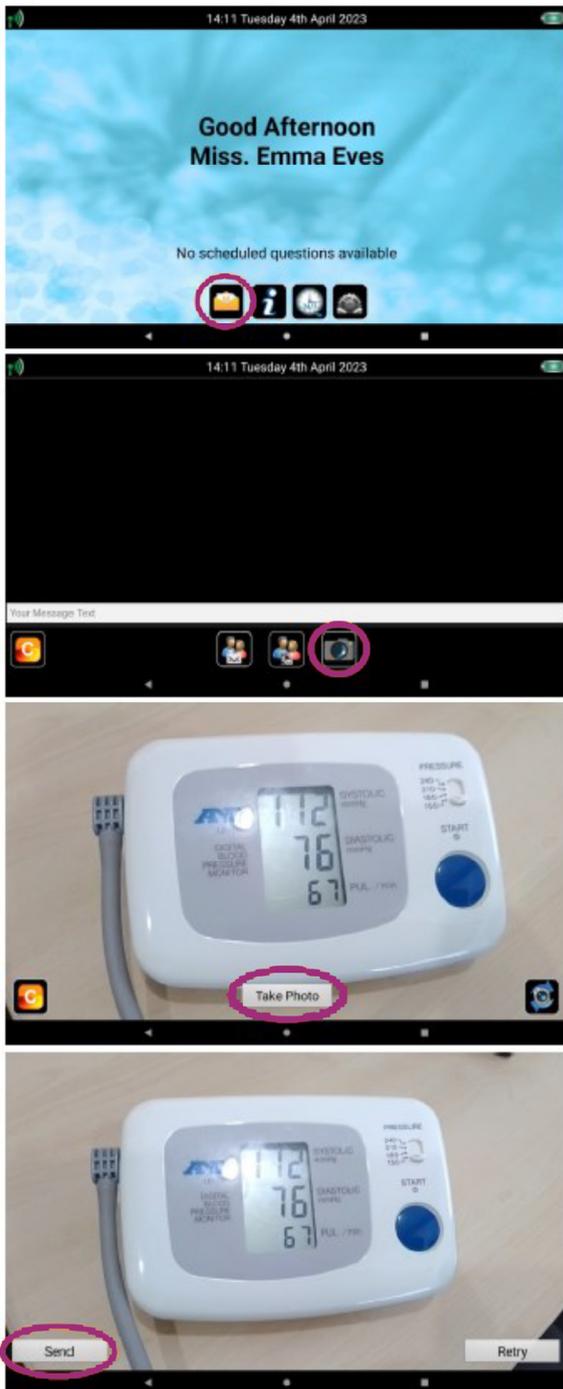



For technical assistance call 01372 899031

**Graphnet**   
Transforming Care

Docobo is a wholly owned  
subsidiary of Graphnet  
Health Limited  
Docobo  
21 High Street Bookham  
Surrey KT23 4AA

Graphnet Health Limited  
Ground Floor  
Building 5 Caldecotte Lake Drive  
Caldecotte  
Milton Keynes  
Buckinghamshire MK7 8LE

T +44 (0)3330 771 988  
E [info@graphnethealth.com](mailto:info@graphnethealth.com)  
[www.graphnethealth.com](http://www.graphnethealth.com)